



MORETZ TRIMMING BLOCK

Warnings / Precautions	Cold soak in glutaraldehyde, chlorine, or ammonium solutions, or dry heat sterilization is not recommended as damage to the block may occur.			
Limitations	Discard any product that shows signs of damage or wear.			
INSTRUCTIONS				
Point of Use	This product is provided non-sterile and must be cleaned and sterilized before the first use and any reuse. Rinse block immediately after use in distilled or demineralized water			
Containment and Transportation	No particular requirements It is recommended that instruments are reprocessed as soon as is practical following use.			
Preparation for decontamination	No particular requirements			
Cleaning: Automated (Do NOT use ultrasonic washer)	Remove instruments and equipment from any sterilization trays before placing into washer baskets. Orient devices following recommendations of washer/disinfector manufacturers. Use alkaline or neutral pH detergent recommended by washer/disinfector or detergent manufacturers. These products have been validated for effective cleaning using an automatic washer/disinfector cycle consisting of a minimum 44 minutes total time, including a pre-wash, main wash & rinse, and thermal rinse. The thermal rinse shall be at least 10 minutes long at a minimum temperature of 60°C.			
Cleaning: Manual	Soak in lukewarm (less than 43°C), mild (pH 7.0 - 8.5), enzymatic detergent and deionized water for a minimum of 2 minutes, then clean ultrasonically in a lukewarm (less than 43°C), mild (pH 7.0 - 8.5), detergent and deionized water for at least 30 seconds. Note: When using an ultrasonic cleaner or a spray washing machine, follow the manufacturer's recommendations, particularly with regard to articulated instruments and positioning of instruments.			
Disinfection	No particular requirements			
Packaging	No particular requirements			
Sterilization (Temperatures are minimum required, times are minimum required)	Cycle:	Gravity	Pre-vac	
	Temperature:	121°C	134°C	
	Time:	30 min	3 min	
	Drying:	8 min or until visibly dry		
	EtO: Not validated		STERRAD Sterilization: Not validated	
Maintenance, Inspection and Testing	After cleaning and sterilization, discard any product that shows signs of damage or wear.			
Storage	No particular requirements			
Additional Information	None			

Note: The instructions provided above have been validated by the manufacturer as being CAPABLE of preparing the product for re-use. It remains the responsibility of the processor to ensure that the reprocessing as actually performed using equipment, materials and personnel in the reprocessing facility achieve the desired result. This normally requires validation and routine monitoring of the process.